HEARTSTART FR_x

INFANT/CHILD KEY

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KINDERSCHLÜSSEL

BABY-/KINDERSLEUTEL

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ENGLISH: About This Edition

The information in this guide applies to the Infant/Child Key 989803139311 for the FRx defibrillator. This information is subject to change. Please contact Philips at www.philips.com/productdocs for information on revisions.

Edition History

Edition 17

Publication date: February 2013

Publication number: 453564421141

© 2013 Philips Electronics North America Corp. No part of this publication may be reproduced, transmitted, transcribed, stored in a retrieval system or translated into any human or computer language in any form by any means without the consent of the copyright holder.

Unauthorized copying of this publication may not only infringe copyright but also reduce the ability of Philips Medical Systems to provide accurate and up-to-date information to users and operators alike.

Authorized European Representative

Philips Medizin Systeme Boeblingen GmbH
Hewlett-Packard Strasse 2
71034 Boeblingen, Germany
(+49) 7031-463-2254

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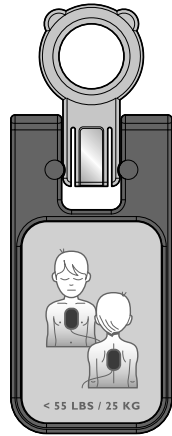
Philips Medical Systems
22100 Bothell Everett Highway
Bothell, WA 98021-8431, USA

INTENTIONALLY BLANK
INTENTIONNELLEMENT LAISSÉ EN BLANC
DIESE SEITE BLEIBT FREI.
OPZETTELJK ONBEDRUKT GELATEN
ESTE ESPACIO SE HA DEJADO EN BLANCO DELIBERADAMENTE
PAGINA LASCIATA INTENZIONALMENTE VUOTA
PÁGINA DEIXADA EM BRANCO INTENCIONALMENTE
SIDAN ÄR AVSIKTLIGEN TOM
SKAL VÆRE TOM
JÄTETTY TARKOITUKSELLA TYHJÄKSI
DENNE SIDE ER TILSIGTET TOM
TĘ STRONĘ UMYŚLNIE POZOSTAWIONO PUSTĄ
ZÁMĚRNĚ PONECHÁNO PRÁZDNÉ
СТРАНИЦА ОСТАВЛЕНА ПУСТОЙ НАМЕРЕННО
ΣΕΛΙΔΑ ΣΚΟΠΙΜΑ ΚΕΝΗ
BOŞ BIRAKILMIŞTIR
刻意留白
刻意留空
このページは空白です
이 페이지는 공백 페이지입니다
หน้านี้ไม่มีข้อมูล
فارغ عن قصد

Infant/Child Key, front.

98980313931 | INFANT/CHILD KEY

The Infant/Child Key is for use *only* with the HeartStart FRx Defibrillator to treat an infant or child who is under 55 pounds (25 kg) or 8 years old. If you are uncertain of the child's exact weight or age, apply the defibrillator *without* the Infant/Child Key. For instructions on using the FRx Defibrillator, refer to the FRx Defibrillator *Owner's Manual* or *Quick Reference Guide*.

WARNING: Infants and children more often become unresponsive due to breathing or other issues than to heart problems.

- Give one minute of infant/child CPR while a bystander calls 911/EMS.
- If no bystander is available, call EMS *after* giving CPR for 1 minute.
- Turn on the HeartStart FRx Defibrillator and install the Infant/Child Key. Alternatively, follow your local protocol.

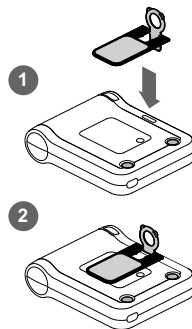
With the Infant/Child Key inserted, the HeartStart FRx will announce "Infant/Child Mode," automatically reduce the defibrillation energy, and provide optional infant/child CPR coaching.

No special defibrillator pads are required to use the HeartStart FRx to treat an infant or child; the HeartStart SMART Pads II provided with the defibrillator are designed for both adult and infant/child use.

NOTE: It does not matter whether you insert the Infant/Child Key before or immediately after turning on the FRx. However, the Key should be inserted *before* placing the pads on the patient. If the Infant/Child Key is removed during use, the FRx will announce “Adult Mode.” Any shocks delivered will be adult energy and optional CPR coaching will be for adults.

TO USE THE KEY


Remove the Key from its pocket in the carry case.

Insert the Key.

Insert the Key into the slot at the top center of the front panel of the FRx Defibrillator. Insert the Key as shown on the back of the Key. The pink portion of the Key pivots (1) and fits into the slot (2), with the front of the Key lying flat on the surface of the FRx so the pediatric pads placement diagram is visible.

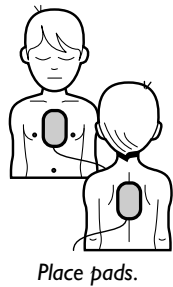The pads icons on the Key start flashing when the defibrillator is turned on and the Key is inserted. **IMPORTANT: Use the illustration on the Infant/Child Key as a guide to correct pads placement.** This placement is different from the standard placement illustrated on the pads for adults and older children.


Place pads, one on the chest and one on the back. *It does not matter which pad is placed on the chest or the back.* Once pads are placed, the pads icons stop flashing. Follow the FRx Defibrillator's voice instructions. It will talk you through each step of using it.

CAUTION: Do not leave the Infant/Child Key inserted in the FRx Defibrillator when it is not being used in an emergency. This will cause the FRx to start chirping. In addition, the lower energy provided when the Key is inserted is only appropriate for treating patients under 55 pounds or 8 years old.

TRAINING. The Infant/Child Key can be used with HeartStart Training Pads II to create a training scenario for treating infants and children. CPR for infants and children is different than for adults. It is strongly recommended that anyone who may use the Infant/Child Key be trained in infant/child CPR.

STORAGE. Keep the Infant/Child Key in its pocket in the carry case.

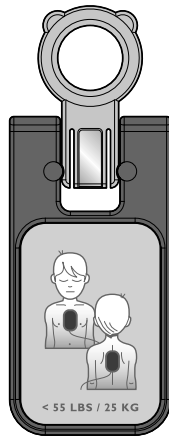MAINTENANCE. The Infant/Child Key can be cleaned with a soft cloth dampened in soapy water, chlorine bleach (2 tablespoons per quart or liter of water), an ammonia-based cleaner, or isopropyl alcohol (70% solution).

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© 2013 Philips Electronics North America Corp. Kein Teil dieser Publikation darf ohne Zustimmung der Copyright-Inhaber in irgendeiner Form oder auf irgendeine Weise reproduziert, übertragen, kopiert, gespeichert oder in eine menschliche oder Computersprache übersetzt werden.
Niets uit deze uitgave mag in enige vorm of op enige manier worden verveelvoudigd, verzonden, getranscribeerd, opgeslagen in een geautomatiseerd gegevensbestand of vertaald in menselijke of computertaal zonder toestemming van de houder van het copyright.
Philips Healthcare, a Division of Philips Electronics Ltd.
Philips Electronics Hong Kong Ltd.