



ROW'S XL
EB04.008 / 010

SUGGESTED USE | MANUEL D'INSTRUCTION

EB04.010
Red colour.
Couleur Rouge



EB04.008
Blue colour.
Couleur Bleu



- 1 Comfortable padded shoulder strap makes for easy carrying.
- 2 Full width PVC pocket to hold and display identifying cards and documents.
- 3 If internal and external surfaces are sanitisable and liquid impermeable.
- 4 Handle for easy transport
- 5 Thermometer with external display allows temperature monitoring.

- 1 Bandoulière pour faciliter le transport.
- 2 Pochette frontale transparente pour identification du contenu.
- 3 Matière adéquate pour une désinfection et un nettoyage facile.
- 4 Poignée de transport.
- 5 Thermomètre extérieur qui contrôle la température intérieure.



MAIN COMPARTMENT | COMPARTIMENTO PRINCIPAL

EB04.008
Blue colour.
Couleur Bleu



- 1 Includes two foam sample blocks, with sufficient capacity for over 80 collection tubes. 1 2 porte-échantillons inclus.

BATTERY REPLACEMENT | CHANGEMENT DES PILES

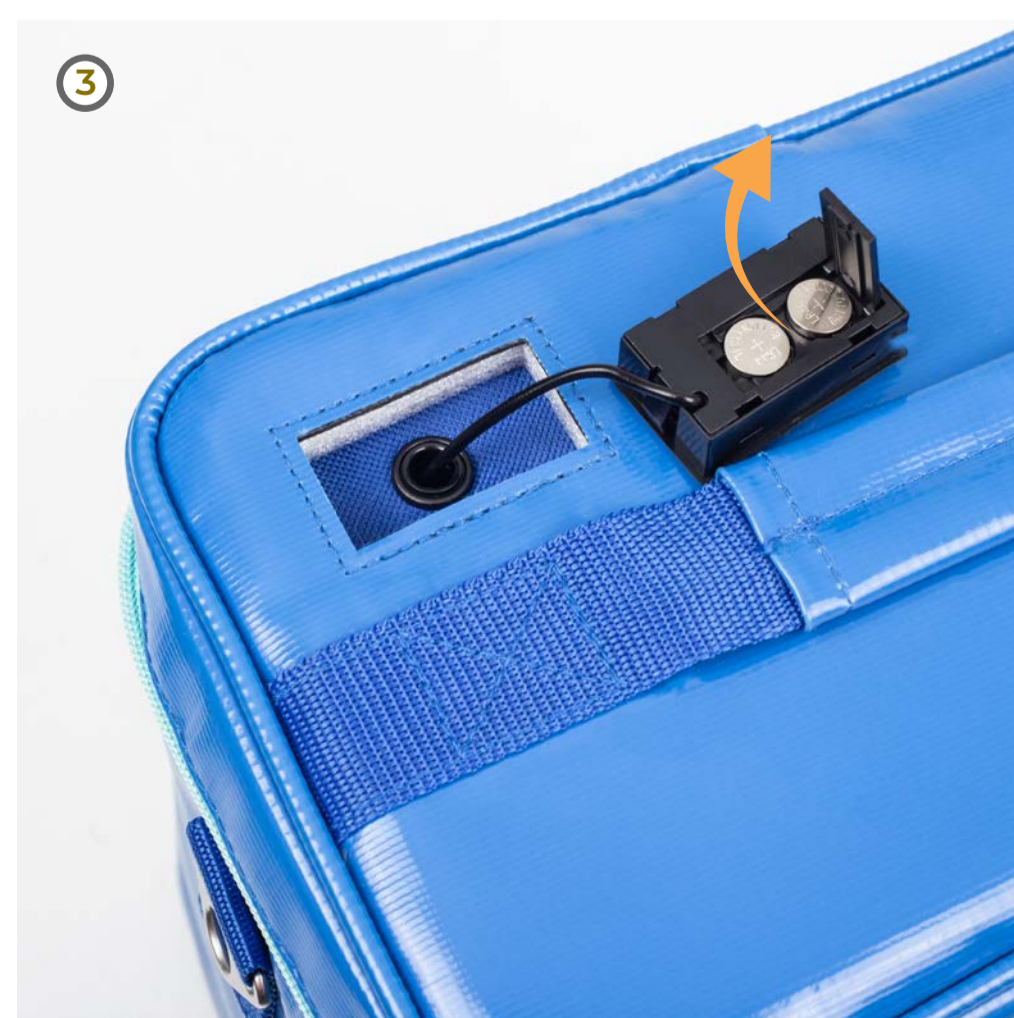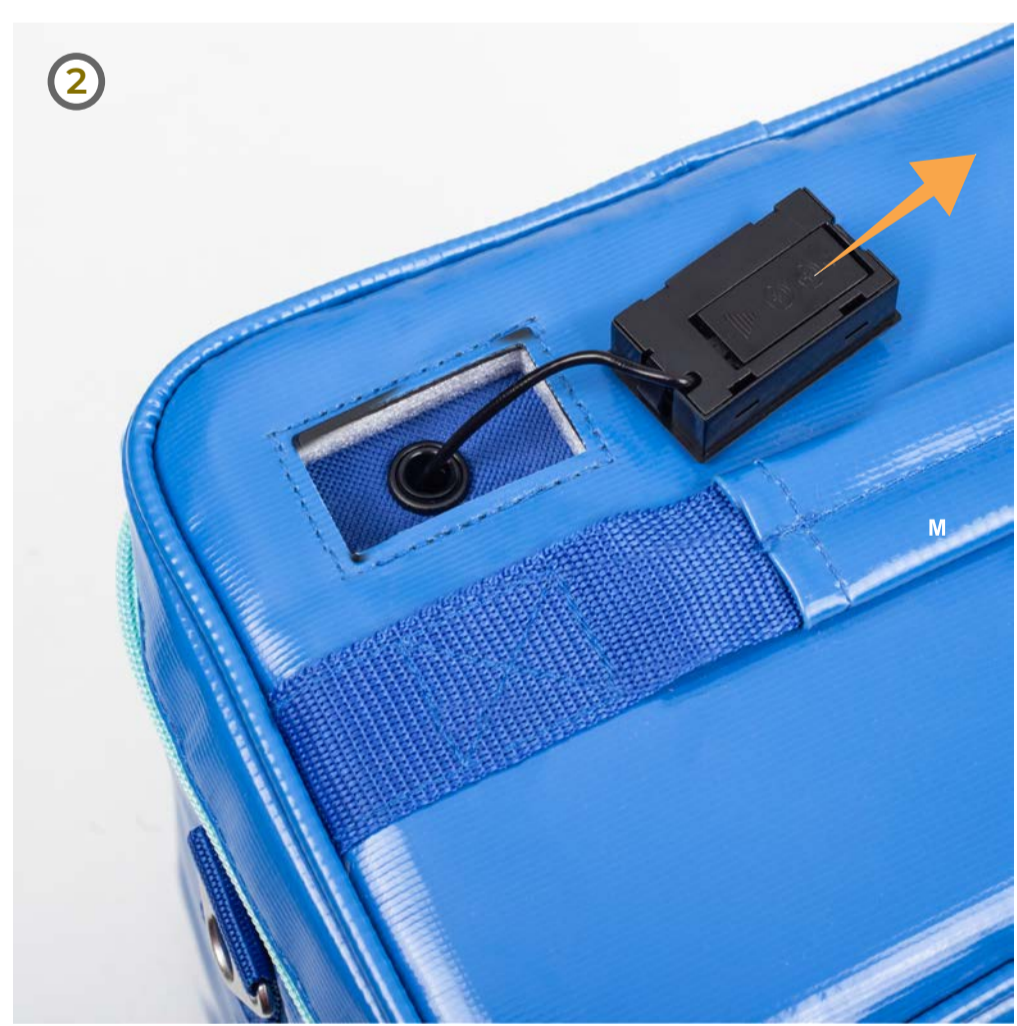

- 1 Remove the LCD module.
- 2 Slide the battery door in the direction indicated by the arrow.
- 3 Open the compartment and replace the exhausted batteries.
- 4 Close the hatch again and insert the module again until it clicks into place.

- 1 Extraire le module LCD en poussant depuis l'intérieur, le module LCD vers l'extérieur.
- 2 Ouvrir le compartiment à pile dans le sens indiqué par la flèche photo 3.
- 3 Changer les piles, refermer le compartiment.
- 4 Remettre le module LCD dans son emplacement initial.