Dual flow gastric aspiration tube & neosump

REF 340 - 1340 - 5382 - 80340

English

DUAL FLOW GASTRIC ASPIRATION TUBE & NEOSUMP

Dual flow gastric aspiration tube are indicated to prevent the accumulation of gastric secretions and air allowing the stomach decompression by continuous suctioning without any injury to the stomach mucosa.

INDICATIONS

- Prevent gastric distension
- Avoid inhalation for ventilated patients
- Avoid vomiting
- Avoids sutures tension post-surgery

CONTRAINDICATIONS

All the following contraindications are relative:

- Facial fractures (orogastric intubation may be a better option)
- Esophageal obstruction, stenosis, alkali ingestion
- Esophageal varices
- Comatose patients without protected airways
- Penetrating neck wounds
- Choanal atresia
- Recent Oropharyngeal, nasal, or Gastric surgery
- Zenker's diverticule
- Severe coagulopathy

COMPLICATIONS/ SIDE EFFECTS: DURING INSERTION

- Tube misplacement (in respiratory tree or pleural space)
- Esophageal perforation

AFTER INSERTION

- Patient discomfort
- Tube obstruction
- Tube displacement
- Haemorrhage (epistaxis)
- Mucous alteration (sinusitis, nasal tissue erosion, tracheo-oesophagus fistula)
- Nasogastric tube syndrome (postcroico ulceration and infection with a reported life-threatening complication with laryngeal and upper airway obstruction and vocal cord paralysis)
- Dual flow gastric aspiration tube are available in 60, 90 and 120 cm.
- Dual flow gastric aspiration tubes & Neosump (code: 340/80340/5382) are made of PVC.
- Dual flow gastric aspiration tubes code 1340 are made of PUR.
- Should the tube have to be used more than 5 days, we recommend to use a PUR tube.
- Dual flow gastric aspiration tubes made of PVC can be used up to 5 days.
- Dual flow gastric aspiration tubes made of PUR can be used up to 4 weeks.

POPULATION

Dual flow gastric aspiration tubes are designed for neonates, paediatrics and adult patients.

Gastric suction tubes size 6 Fr are mainly used for neonatology.

Gastric suction from 8 Fr is mainly used for paediatrics.

Gastric suction of 10 Fr or more are mainly used for adult patients.

USERS

In general, it should be noted that the dual flow gastric aspiration tube is used by well-trained physicians strictly follow the hospital protocol.

TUBE PLACEMENT

Gather all equipment required:

- Device to clean the nostrils.
- Disposable gloves
- Fixation device
- If required a lubricant
- Syringe adapted to the gastric aspiration tube
- Tube: for size and material selection, refer to doctor prescriptions.

INFORM THE PATIENT (IF THIS IS POSSIBLE).

-For unconscious patients, secure the airways with IT tubes

PLACEMENT OF THE GASTRIC SUCTION TUBE

For oral or nasal placement, measure the length of tube to be inserted according to standard protocols. Lubricate the suction tube.

Introduce the dual flow gastric aspiration tube in one of the nostrils (or the mouth) and move the tube carefully towards the pharynx. Orientate the tube towards esophagus.

Once the correct insertion length has been reached, stop the advancement of the tube.

Oral placement only for 30 Fr & 36 Fr tubes, (codes 340.30 and 340.36).

Oral or nasal placement for 6 Fr to 21 Fr tubes. Angle patient head downwards to close the trachea and open esophagus.

Proceed slowly to avoid the risk of patient vomiting.

FIXATION OF THE DUAL FLOW GASTRIC ASPIRATION TUBE

The tube may be fixed to the nose using an hypoallergenic tape.

Attachment should enable the tube to adopt a natural position without excessive pressure on the nostrils.

CHECK CORRECT POSITIONING

Check tube placement before aspiration.

SUCTION

Connect main lumen hub to the vacuum source through the double cone straight adaptor included in the packaging. The lateral lumen is the venting lumen which has to be maintained open.

- For the special dual flow gastric aspiration tube made of PVC (codes: 80340): you can perform a syringe suctioning thanks to the luer female adaptor available in the packaging.

- For the Nutrisafe2 dual flow gastric aspiration tube made of PVC (codes: 340.xx2): you can perform a suction using a Nutrisafe2 syringe.

To avoid any gastro-esophageal reflux through the venting lumen place the hub of the venting lumen over the patient's stomach level and the container for gastric residues under the stomach level.

The gastric suctioning via the dual flow gastric aspiration tube is performed thanks to a vacuum source between 100 to 400 mbars.

After the suction, in order to avoid any leakage through the tube between two uses, the aspiration lumen (main lumen) and the venting lumen (the secondary lumen) are connected to each other.

PRECAUTION FOR USE

Caution: Federal law (USA) restricts this device to sale by or on the order of a physician.

An excessive traction of the device could induce damages and / or displace the tube.

The rules of good hygiene should be followed at each step of the placement of the tube.

The gastric suction tube placement is verified before each use (in any case follow the protocols of your institution and the medical prescriptions for the tube size and material selection).

Strictly follow the protocol of your institution to clean the tube between two uses.

Store protected from heat, moisture and sunlight.

This device is not made with dry or natural rubber latex. (Except for the codes 340.30 and 340.36)

Re-use of this device may change its mechanical or biological features and may cause device failure, allergic reactions or bacterial infections.

To dispose of, place the contaminated items in the appropriate receptacle.

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Once the correct insertion length has been reached,

stop the advancement of the tube.

Oral placement only for 30 Fr & 36 Fr tubes, (codes 340.30 and 340.36).

Oral or nasal placement for 6 Fr to 21 Fr tubes. Angle patient head downwards to close the trachea and open esophagus.

Proceed slowly to avoid the risk of patient vomiting.

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