



# Filter - B/V Filter

**REF** 70530

**LOT**



Storage temp: 15°C ~ 25°C  
Storage relative humidity: <85%



Date of Manufacture:  
YYYY-MM



Use by:  
YYYY-MM

Contains: one ( 1 ) piece  
Intended to reduce and filter the transmission of microbes and other particulate matter in breathing systems.

- \*Electrosatic media
- \*High efficiency with low resistance filter
- \*Filter efficiency: 99.99% (BFE) , 99.9% (VFE)
- \*Standard coaxial 22F for inlet port and 22M/15F for patient port

### **Caution:**

- \* Intended for use by qualified personnel trained.
- \* For best performance, use within 3 years from the date of manufacture.
- \* Store away from heat and light.
- \* Do not sterilize.
- \* The device is single use and must not be re-used. Reuse may cause cross infection and reduce product reliability and functionality.

### **Instruction of use:**

- \* Take the filter out from package, make sure the filter inside is in good condition without damage.
- \* Connect the filter to the tube or parts.
- \* Make sure the connections are sealed and tight.
- \* Start to use

Manufactured by  
HSINER Co.,Ltd.  
312, Jhongsan Rd., Shengang Dist,  
Taichung City 429, Taiwan  
Tel: 886-4-25152480  
Fax: 886-4-25152482  
Email:[sales@hsiner.com](mailto:sales@hsiner.com)  
Web Site:<http://www.hsiner.com>

**EC REP** EU Representative  
mdi Europa GmbH  
Langenhagener Str. 71,  
D-30855 Langenhagen  
Tel: +49 511 3908 9530  
Fax: +49 511 3908 9539  
<http://www.mdi-europa.com>

**CE**  
2460

AT0017053010  
Rev.10 13/07/20