



Aerosol Therapy- Nebulizing Set/Kit Adult



Storage temp: 15°C ~ 25°C

Storage relative humidity: <85%

REF 30142

LOT



Date of Manufacture:
YYYY-MM



Use by:
YYYY-MM

Contains: one (1) ea

Includes: Aerosol mask, jet nebulizer, 2 meter oxygen tube

Designed for using on aerosol therapy

Material: polyvinyl chloride, polystyrene, polypropylene

Caution:

- * Nebulizer must only be used by persons who have received adequate training and instruction from qualified medical personnel.
- * For the best performance, use within 3 years from the date of manufacture
- * Store away from heat and light
- * The device is single use and must not be re-used. Reuse may cause cross infection and reduce product reliability and functionality.

Instruction for use:

1. Connect the oxygen tube onto nebulizer and aerosol mask, then to a source of gas.
2. Be sure connections are secured.
3. Tighten elastic about head and over ears.
4. If there is concern of mask failure, replace mask immediately.

 Manufactured by
HSINER Co.,Ltd.
312, Jhongshan Rd., Shengang Dist,
Taichung City 429, Taiwan
Tel: 886-4-25152480
Fax: 886-4-25152482
Email: sales@hsiner.com
Web Site: <http://www.hsiner.com>

EC REP EU Representative
mdi Europa GmbH
Langenhagener Str. 71,
D-30855 Langenhagen
Tel: +49 511 3908 9530
Fax: +49 511 3908 9539
<http://www.mdi-europa.com>

CE
2460

AT0013014208
Rev.08 25/03/21