4BC

Kit Hémorragies

CONTENU:

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Dispositif non stérile

ALWAYS USE GLOVES

**Emergency bandage FIRST CARE
Trauma Wound Dressing 10 cm**

<p>1</p>  <p>Place pad on wound & wrap the elastic bandage around limb or body part</p>	<p>2</p>  <p>Insert elastic bandage into pressure applicator</p>	<p>3</p>  <p>Tighten elastic bandage and...</p>
<p>4</p>  <p>...pull back, forcing pressure applicator down onto pad</p>	<p>5</p>  <p>Wrap elastic bandage tightly over pressure applicator. Wrap over all edges of pad</p>	<p>6</p>  <p>Secure hooking end of closure bar into elastic bandage</p>
<p>7</p> <p>To remove Emergency Bandage, free hooking ends from elastic bandage</p>		

Sterile single use device (see packing)

Hemostatic Gauze CELOX RAPID

FOR USE BY TRAINED EMERGENCY RESPONDERS
FOR THE TREATMENT OF VITAL RISK BLEEDING







<p>1</p>  <p>Tear open pack. Take out the CELOX RAPID and take one end of gauze</p>	<p>2</p>  <p>Tightly pack the unfolding CELOX RAPID directly to the source of the bleeding. Pack remaining wound cavity with CELOX RAPID or standard gauze.</p>	
<p>3</p>  <p>Apply FIRM pressure directly to the wound for at least 1 minute or until bleeding stops</p>	<p>4 . Excess CELOX RAPID can be torn or cut if necessary</p> <p>Wrap and tie with a bandage so as to maintain pressure on the wound</p>	
<p>5. Discard any remaining CELOX RAPID</p>	<p>6. Transfer patient to medical facility as soon as possible</p>	<p>7. Show empty pack to medical personnel</p>

ATTENTION MEDICAL FACILITY PERSONNEL.

1. Physically remove CELOX RAPID from the wound and any loose surface granules.
2. Fully flood entire wound area with sterile saline irrigation solution. Proceed with normal irrigation and / or suction. Ensure all products is removed from the wound prior to initiate wound treatment.

Warning : Device not intended for surgical use. Do not eat. If ingested drink water to avoid any discomfort. Do not apply over eyes. If eye irritation occurs flush with water for 5 minutes. Sterile unless the pouch is damaged or opened. A loss of sterility is a potential risk of infection. Reuse could lead to risks of infection and reduced performance. Contains chitosan from shellfish. Shellfish allergy studies show no adverse reaction. Clinical data are available at MedTrade Products Ltd.

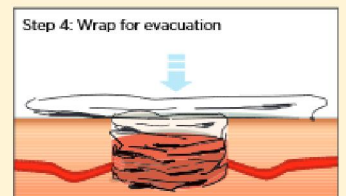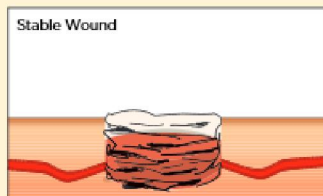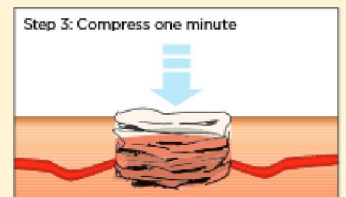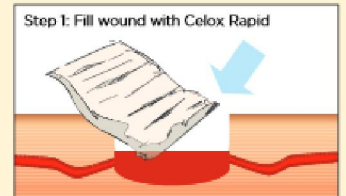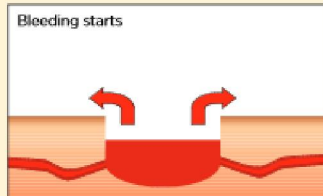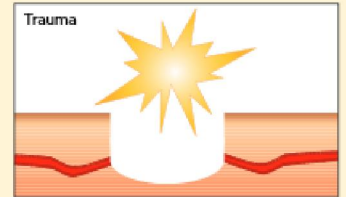Sterile single use device (see packing)

SOF-TT Tourniquet

SOF Tactical Tourniquet SOF-TT

For control of severe bleeding of upper or lower limbs


Store the Tourniquet with the buckle connected and enough slack to slip the tourniquet loop over a arm or a leg.

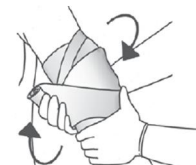Instruction for use to apply the Tourniquet		Applying the Tourniquet if unable to slide over arm or leg Follow steps 1b-3b	
1  Slide over arm or leg	2  Pull tail tight	1b  Unbuckle	2b  Go around arm or leg
3  Twist handle	4  Lock in triangle	3b  Connect buckle, then go to step 2	
5  Note time			

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